

DreamStation 2 CPAP Advanced and Auto CPAP Advanced Homecare provider competency check-list

Assessed:

Assessor:

Date:

Goals	Date goals were met	Assessor
Principles of operation		
1. Describe the intended use of the device: Designated patient populations Describe appropriate delivery environments		
2. Identify and demonstrate the components of the DreamStation 2: Mode (CPAP or Auto CPAP) Opti-Start* CPAP-Check FLEX (C-Flex+ and A-Flex)* EZ-Start Auto-Trial Ramp Plus *Adjustable in Care Orchestrator		
Setup and operation		
1. Identify and demonstrate how to replace the circuit components of the DreamStation 2 Advanced system: 6' tubing Power supply Optional bacterial filter Ultra-fine and pollen filters Integrated water chamber, seal and lid		
2. Demonstrate the steps to access and utilize Provider Mode: Power the device Configure device to access Provider Mode Demonstrate mode selection Demonstrate setting selections Demonstrate access to detailed patient therapy data Demonstrate access to additional device information		

Goals	Date goals were met	Assessor
The clinician will be able to:		
1. Demonstrate how to utilize Provider Mode to verify compliance: <div> <div>AHI</div> <div>Machine hours</div> <div>90% pressure</div> <div>Days>4</div> <div>Mask fit %</div> </div>		
2. Demonstrate navigation of Patient Mode: <div> <div>Powering the device</div> <div>Viewing therapy information</div> <div>Initiating therapy</div> <div>Viewing and adjusting</div> <div>Adjusting Ramp Plus Pressure</div> <div>additional patient and</div> <div>Adjusting humidification</div> <div>device settings</div> <div>Adjusting heated tube temperature (if applicable)</div> </div>		
3. Describe the various methods of obtaining data from the device.		
4. Describe the method and issues involved with adding supplemental oxygen.		
Alert conditions and troubleshooting		
1. Demonstrate understanding of device alerts, including possible causes and corrective action. <div> <div>Service required</div> <div>SD card</div> <div>Errors</div> <div>Modem</div> <div>Patient reminder</div> </div>		
Cleaning and care		
1. Demonstrate the method of cleaning the device, interface, and replacement of the filters.		

Assessed signature:

Assessor signature:

